



hydroven® 1 & 3

## Garment Measuring Guide

**HUNTLEIGH**

A MEMBER OF THE ARJO FAMILY


# Garment Measuring Guide

The following guide is intended as an aid to help the user select the correct size garment.

The treatment type will determine the garment type e.g. hydroven® 1 for uniform compression or hydroven® 3 for graduated pressure.

When determining the correct size garment:

- Remove any bandaging, stockings or bulky dressings
- Measure the points as shown on the diagram below
- DO NOT pull the measuring tape tight
- Make sure the garment extends beyond the area under treatment.

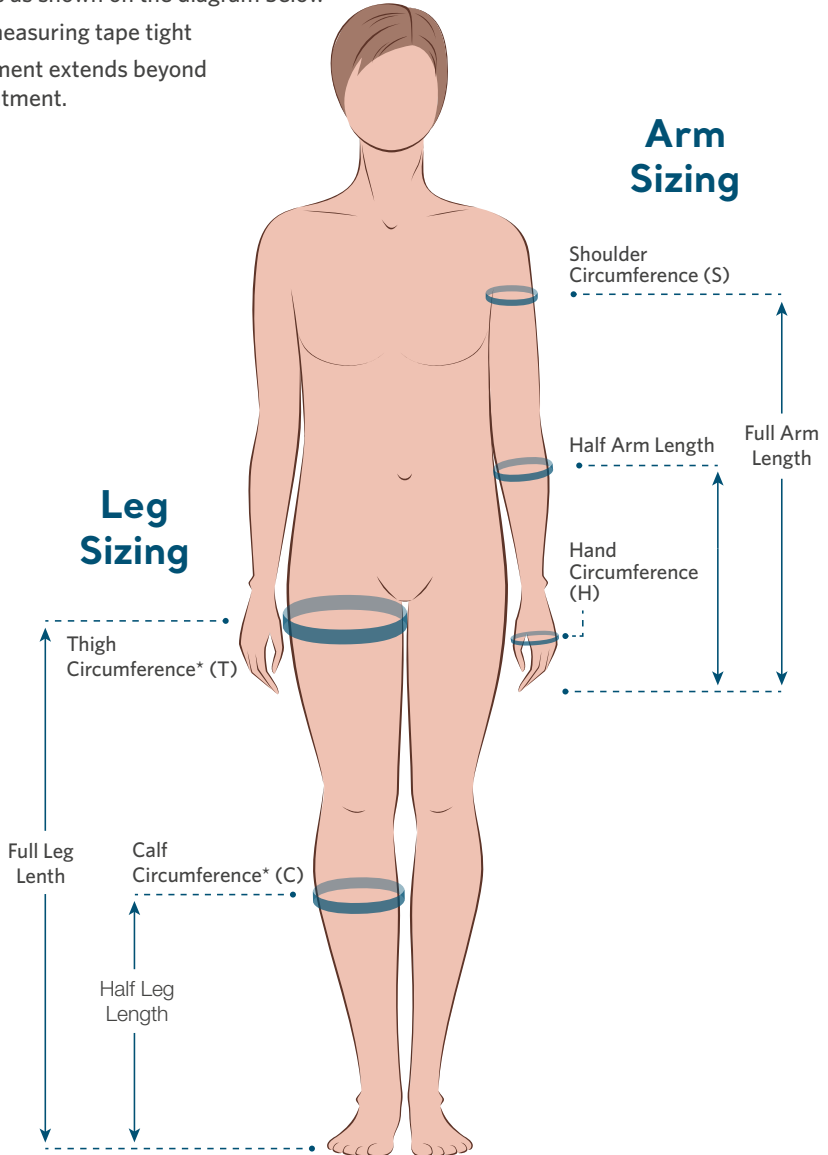

\*Measurements should be made at the widest point on the calf and thigh.

## Hydroven 3 Arm

Three Chamber Arm Garments



Order Code	Type	Length (L)	Shoulder Circumference (S)	Hand Circumference (H)
5103A68	Full Arm	68cm	62cm	44cm
5103A78	Full Arm	78cm	62cm	44cm

## Hydroven 3 Leg

Three Chamber Leg Garments



Order Code	Type	Length (L)	Thigh Circumference (T)	Calf Circumference (C)
5103L50	Half Leg	50cm	61cm	56cm
5103L66	Full Leg	66cm	64cm	56cm
5103L71	Full Leg	71cm	66cm	56cm
5103L76	Full Leg	76cm	72cm	56cm
5103L84	Full Leg	84cm	72cm	56cm
5103L92	Full Leg	92cm	72cm	56cm

## Hydroven Inserts

Compatible with Hydroven 1 and Hydroven 3 garments



Order Code	Type	Length (L)	Added Circumference Wide End	Added Circumference Narrow End
Leg Inserts				
510LI50	Half Leg Insert	50cm	19cm	14cm
510LI66	Full Leg Insert	66cm	19cm	14cm
510LI71	Full Leg Insert	71cm	19cm	14cm
510LI76	Full Leg Insert	76cm	19cm	14cm
510LI84	Full Leg Insert	84cm	19cm	14cm
510LI92	Full Leg Insert	92cm	19cm	14cm
Arm Inserts				
510AI68	Full Arm Insert	68cm	17cm	12cm
510AI78	Full Arm Insert	78cm	17cm	12cm

# Hydroven 1 Arm

## Single Chamber Arm Garments



Order Code	Type	Length (L)	Shoulder Circumference (S)	Hand Circumference (H)
5101A51	Half Arm	51cm	56cm	44cm
5101A68	Full Arm	68cm	62cm	44cm
5101A78	Full Arm	78cm	62cm	44cm

# Hydroven 1 Leg

## Single Chamber Leg Garments



Order Code	Type	Length (L)	Thigh Circumference (T)	Calf Circumference (C)
5101L50	Half Leg	50cm	61cm	50cm
5101L66	Full Leg	66cm	64cm	54cm
5101L71	Full Leg	71cm	66cm	55cm
5101L76	Full Leg	76cm	72cm	56cm
5101L84	Full Leg	84cm	72cm	57cm
5101L92	Full Leg	92cm	72cm	57cm

As a proud member of the Arjo family, we have been committed to supporting healthcare professionals in improving outcomes and enhancing patient wellbeing since 1979. We do this through our proven solutions for Vascular Assessment & Treatment and Fetal & Patient Monitoring. With innovation and customer satisfaction as our guiding principles, we strive for clinical excellence and improved performance, for life.

**Manufactured and distributed by Huntleigh Healthcare Ltd.**  
 35 Partmanmoor Road, Cardiff, CF24 5HN, United Kingdom  
 T: +44 (0)29 20485885 sales@huntleigh-diagnostics.co.uk  
 www.huntleigh-diagnostics.com

Registered No: 942245 England & Wales. Registered Office:  
 ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, Bedfordshire, LU5 5XF  
 ©Huntleigh Healthcare Limited 2019



**A Member of the Arjo Family**  
 ® and ™ are trademarks of Huntleigh Technology Limited  
 As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

AW: 1001043-2